

## Now you can keep on going

You have already seen and done a lot. But your journey of discovery isn't over yet. Effortless staying power. Live life to the full. That's what Rollz is all about. Which is why we have designed the modern Rollz Motion, a rollator that converts into a transportchair at a flick of the wrist. And of course, just as easily back again. Rollz takes you further.

Be it for an outdoor activity, a long holiday, a treasured exposition or a shopping afternoon, this 2-in-1 rollator inspires you to keep on going without effort. In addition the Rollz Motion folds in a compact package and fits into any car.



2012



reddot design award  
honourable mention 2012



**Triumph Mobility**  
151 Bentley Ave., Unit 4  
Nepean ON  
K2E 6T7  
Phone: 1-855-546-0711  
Fax: 1-866-225-3059  
Email: sales@triumphmobility.com  
[www.triumphmobility.com](http://www.triumphmobility.com)



**Rollator and  
Transportchair  
in one**



## Rollator and Transportchair

The Rollz Motion is an ergonomically designed rollator with a strong but lightweight frame which enables you to walk steadily, upright and comfortably. And if you want to rest along the way, the Rollz Motion converts into a transportchair at a flick of the wrist. And of course, just as easily back again.



### From rollator to transportchair at a flick of the wrist

Before you leave home, click the Rollz transportchair attachment into place on the rollator.

1. Pull the backrest into position in one flowing motion.
2. Turn the handles from the rollator position into the push position, and adjust them to the right height for the person who's going to push.
3. Then click the footrests into place at the front.
4. And continue your journey in comfort and style.

## Product Details

### Compact

The Rollz Motion is easily foldable into a compact package.



### Ease

Cross doorsteps and curbs easily with the threshold aid.



### Stable

Walking tall, steadily and supported between the ergonomic hand grips that are adjustable in height.



### Safe

Innovative drum brakes for safe and balanced braking.

### Convenient

Resting on a comfortable cushioned seat.

### Manoeuvrable

Light steering and easy motion with the manoeuvrable front wheels.

### Comfortable

Comfortable ride and natural suspension thanks to the foam-filled rear tyres.

## Facts & Figures Rollz Motion

### Weight

Rollz Motion: 10.6 kg or 23.7 lb

Rollz transportchair package: 3.7 kg or 8.16 lb

### User

Maximum weight: 125 kg or 275.58 lb

Minimum and maximum height: 1.60m to 1.90m or 63 in to 74.8 in

### Dimensions

Rollz Motion:

Unfolded HxWxL: 86-98 x 65.5 x 63 cm or 33.9-40.2 x 25.8 x 24.8 in

Folded HxWxL: 86-98 x 29 x 63 cm or 33.9-40.2 x 11.4 x 24.8 in

Handle Height Range: 86-98 cm or 33.9-38.6 in

Rollator seat:

HxWxL: 55 x 45 x 20 cm or 21.7 x 17.7 x 7,8 in

Transportchair seat:

HxWxL: 55 x 45 x 38.5 cm or 21.7 x 17.7 x 15.2 in

Distance between handles: 45 cm or 17.7 in

### Accessories

Rollz Shopper

Rollz cup holder

Rollz cane holder

Rollz Motion is available in four colours:

orange, ice-blue, cool grey

